


Cathétérisme - Pathfinder

1



Relocalisation - Repositioning


2

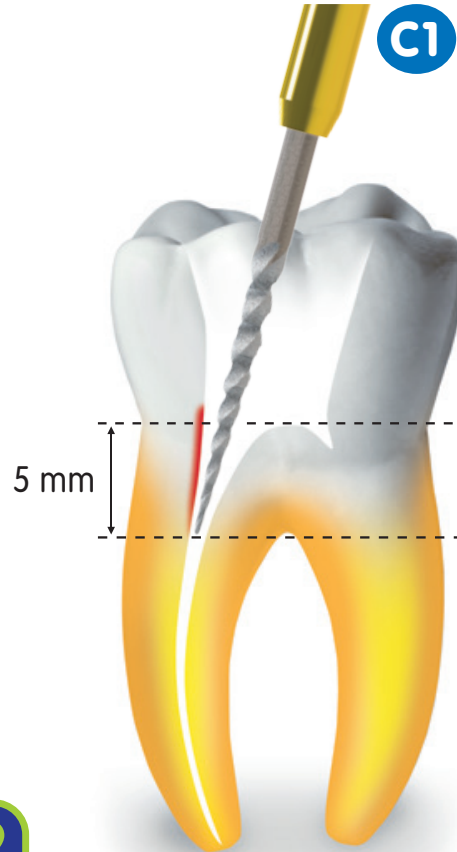

Mise en forme - Shaping

3



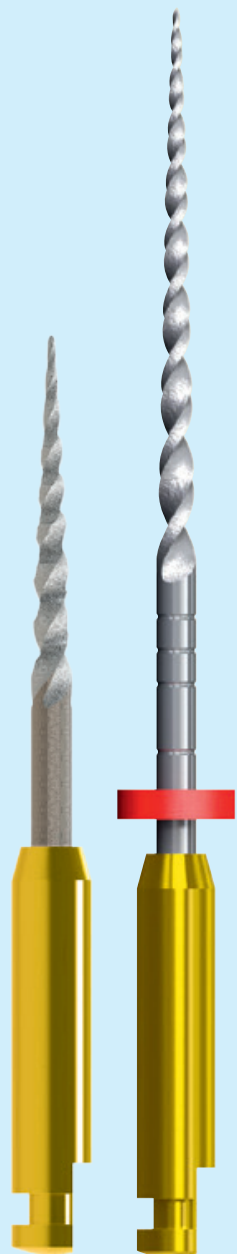

C1

A1



- **Préparer** la cavité d'accès.
- **Déterminer LT** avec une lime K8, K10 ou K15.



- **Prepare** the access cavity.
- **Establish WL** with K8, K10 or K15 file.

- C1** : pour **relocaliser** / **élargir** les entrées canalaire et **éliminer les surplombs** dentinaires.
- Brossage en **appui pariétal**.
 - **Tiers coronaire** uniquement.



- C1** : to **reposition** / **widen** the canal orifice and **remove** dentine **interference**.
- **Circumferential brushing** action.
 - **Coronal third** only.

- A1** : pour **mettre en forme** des tiers **médian** et **apical**.
- Passer **A1** dans le tiers médian avec 3 ou 4 mouvements en **appui pariétal**.
 - Reconfirmer **LT** avec une lime K.
 - Passer **A1** jusqu'à **LT** avec des **mouvements de picotage**.



- A1** : for **shaping** the **middle** and **apical** thirds.
- Take **A1** to the middle third using 3 or 4 **circumferential brushing** actions.
 - Confirm **WL** again with a K file.
 - Take **A1** to **WL** using a **pecking motion**.

Instruments rotatifs en nickel-titane

Rotation continue
Vitesse 300 - 500 tours/mn
Limite de couple 1,5 N. cm
Irriguer constamment
Essuyer fréquemment l'instrument

Nickel-titanium rotary files

Continuous rotation
Speed 300 - 500 rpm
Torque limit 1.5 N. cm
Constantly irrigate
Frequently wipe the file

neoniti

