Dual Functions !!!
Wireless Curing Light & Whitening Accelerator

Dr's Light

User manual

Eng

Kor

Contents

1. Introduction	02
- Contents	
- Features	
- Description	
2. Safety Information	03
- Symbols	
- Notes and reference	
- Installation and Connection	
- Operation	
- Maintenance	
3. List of Components	07
4. Parts and Functions	08
5. Installation	09
6. How to use	13
7. Maintenance	19
8. Trouble Shooting	20
9. Specification	21
10. Accessories	22
11. Warranty	22
12. Quality Assurance	23

1. Introduction

Thank you for purchasing the device. The instructions described below have been designed to provide the necessary information for the successful operation, daily maintenance and safety for the use of this device.

Features

- Dual functions (Curing light + Whitening accelerator)
- High intensity
- Various curing program modes and time setting
- Interchangeable battery
- Color LCD screen
- Unique thermal control system
- Ergonomic design

Description

Polymerization of light-cure catalyst containing dental materials.

2. Safety information

It is important to understand the following information to use this product safely.

Symbols

Warning

Indicates a potentially hazardous situation which, if not avoided, may result in serious injury or death.

Caution

Indicates a potentially hazardous situation which, if not avoided, may result in minor/moderate injury or in damage of property.

Notes and References

Note

that potential malfunctions may occur if the description is ignored.

Reference

Supplementary information to facilitate better operation.

Installation and Connection

Warning

- The adaptor should be disconnected from charger before connect the power cable to prevent an electric shock.
- Keep away from sources of ignition to avoid fire or explosion.
- Protect the handpiece, charger, and battery from any liquid to avoid electric shock or functional error.
- Place the charger on a flat and stable surface. Do not drop handpiece and charger to avoid internal defect that may cause overheating or fire.
- Do not place the electrical cord near a gas burner or sources of heat. If the cord catch on fire, replace to new one to avoid short-circuit.

Caution

- Do not use or leave the unit in areas of high temperature or exposed to direct sunlight. Internal circuit may cause overheat or fire.
- Securely connect the power plug to the outlet with dust-free.
- Do not touch the terminal areas of charger or battery with fingers or conductive objects such as metal wire or safety-pin. It may electric shock or fire.
- Do not use other charger. It many cause fire or device malfunction.

Warning

- Do not plug the power cord with wet hands. This may result in an electric shock.
- In case of fluid leak from battery or partial discoloration / deformation of the handpiece, immediately stop use.
- Place the battery on correct direction when charging. Wrong directed placing may cause malfunction, explosion, or fire.
- In case partial discoloration of the charger, immediately disconnect the power plug and repair the charger.
- In case battery fluid contacts the eye, immediately flush with copious amount of clean water and get medical attention.
- Do not use this device on patients with a history of cataract, retinal damage or allergic reaction to light.
- Do not use this device on a person with a pacemaker as it may cause malfunction of pacemaker.
- Avoid exposing the light on intraoral soft tissue as it may affect the soft tissue.
- Use carefully the guide tip to avoid broken.
- Do not put the battery on the charger upside down.
This may cause an explosion or fire.

Caution

- Disconnect the power cord by grasping the power plug. This may cause injuries or burns.
- Avoid damage or use of unnecessary force on the power cord.
This may cause an electric shock or fire.
- Should battery fluid contact the skin or clothes, immediately wash it off with copious amount water to avoid skin damage.
- Take the battery out of from handpiece, if it won't be used for long time.
- Connect the power plug only when battery is being charged.
Insulation degradation may cause electric shock or fire.
- Do not use this product for any purposes other than dental use.

Maintenance

Warning

- Use only the battery supplied to avoid fluid leak or explosion.
- Never disassemble or overhaul the product. Unusual performance may cause injuries, electric shock, or fire.
- Ensure that charger and battery are always separated from the handpiece prior to cleaning.


Caution


- If the LCD screen malfunctions, remove the battery from the handpiece and reinsert it.
- User must be in charge of operation, maintenance and check.

3. List of components


Please check the contents described below when removing the contents from the original packaging.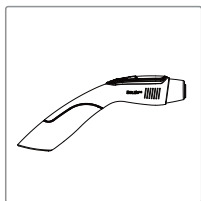


Handpiece

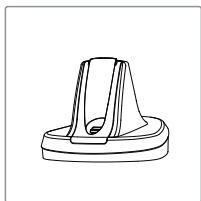
Charger

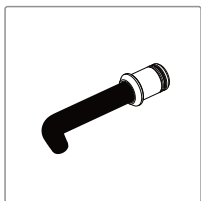
Guide tip 8mm

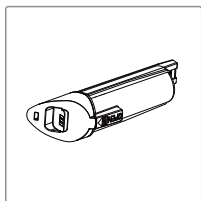
Battery

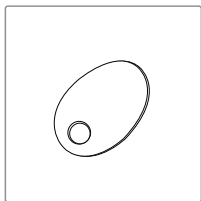Anti-glare Shield

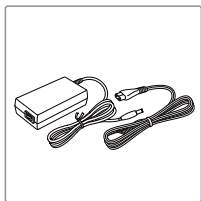
Adapter & Power cable

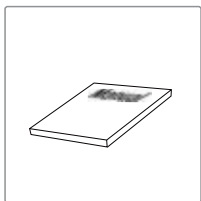
Manual


WA KIT(Optional)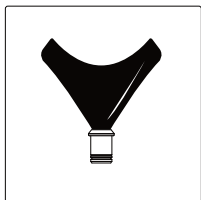


WA Tip

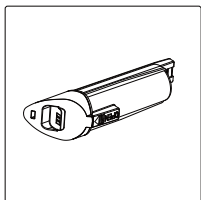Battery

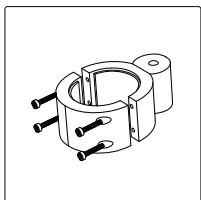
WA Grip

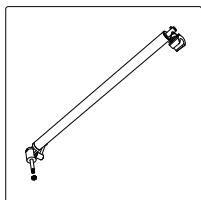WA Arm

4. Parts and functions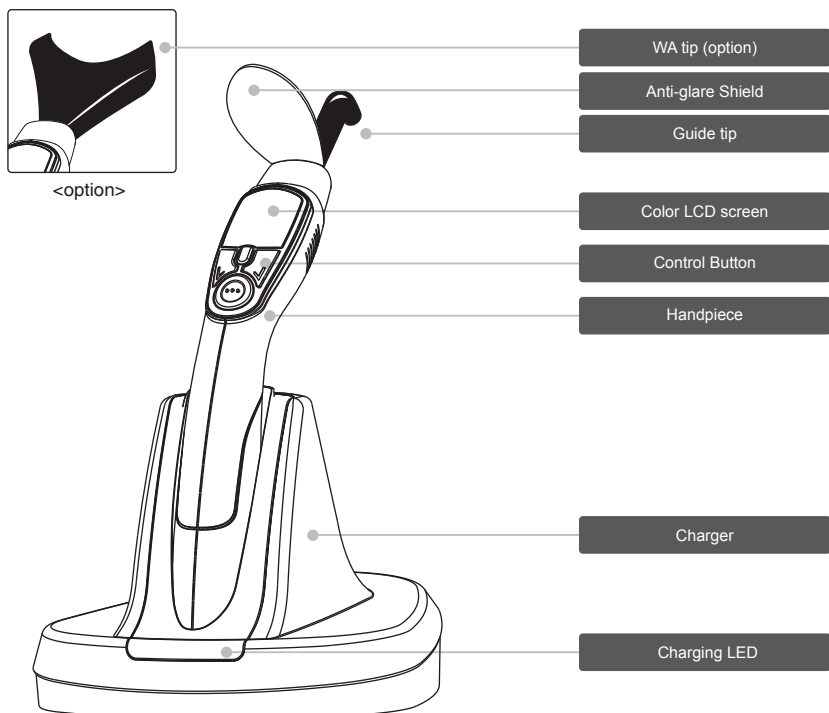

1. Guide tip

- Effectively exposes the LED light

2. Anti-glare shield

- It is placed on the guide tip to protect eyes from exposure to light.

3. Color LCD screen

- Selected mode, exposure time and remaining battery are displayed on the LCD screen in graph mode or text mode.

4. Control button


- T/W button: Short click to turbo mode, long click to WA mode
- Time button
- Mode button
- Operation button

5. Battery

6. Charger


5. Installation

1. Connection between AC adapter and charger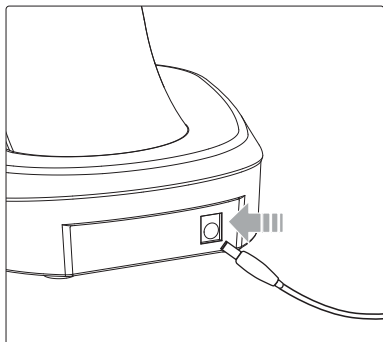

- Securely insert the connector of the AC adapter into the inlet at the back of the charger.

- The AC adapter is able to use with a power outlet of 100V~240V

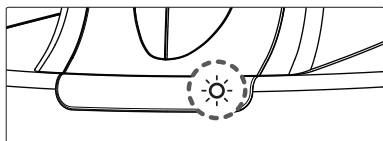
- When the power plug is connected both red and green lights flash instantly which is not a malfunction or an alarm sign.

Note

Use only the original AC adaptor supplied.


2. Inserting battery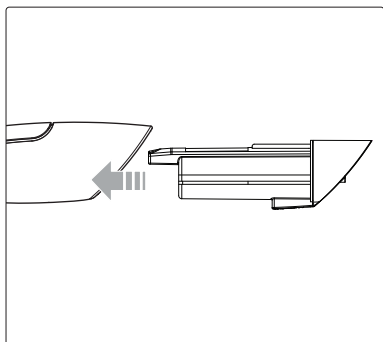

- Carefully insert the battery into the handpiece as directed by the arrow. Avoid forcing the battery pack.


5. Installation

3. Inserting guide tip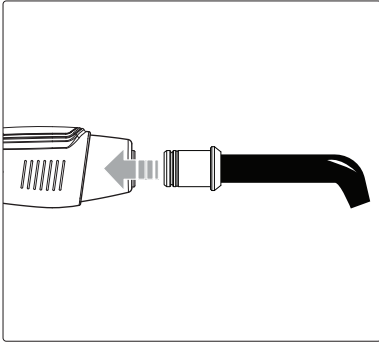


- Slowly insert the guide tip into the handpiece.
Push the guide tip in till it firmly sets into position

4. Placing anti-glare shield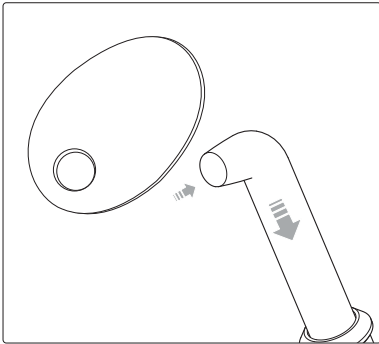

- Place the anti-glare shield on the guide tip.

5. Inserting WA tip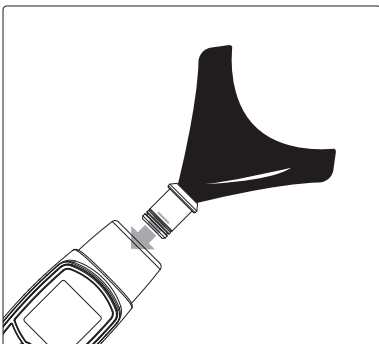


- Slowly insert the WA tip into the handpiece.
Push the WA tip in till it firmly sets into position.


* WA tip is option

5. Installation


6. Charging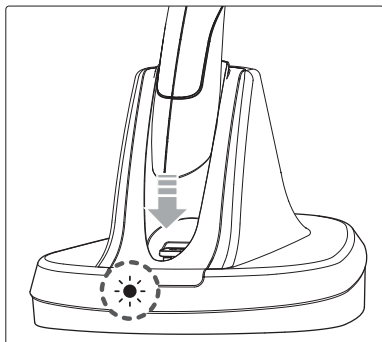

- Insert the handpiece into the vertical slot at the front of the charger. The charger light at the front of the charger will appear.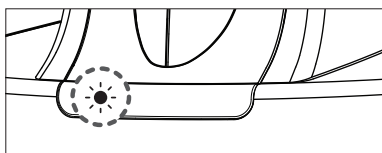

- Red light on: Charging in progress.

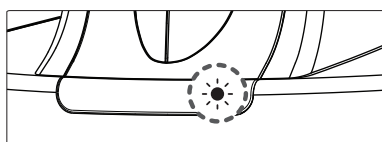
- Green light on: Charging process is complete.

Note

- If the remaining battery is extremely low, red light may flickers while recharging, which is not a malfunction.
- It will take approximately 3 ~ 4 hours before the battery will be fully charged.
- Depending on the volume of the remaining battery, the sign of battery indicator varies.


7. WA KIT installation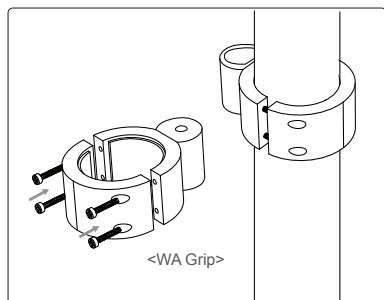

<Installation>


- Mount the WA Grip to unit-chair, and tighten up a bolt by screwdriver tightly.

5. Installation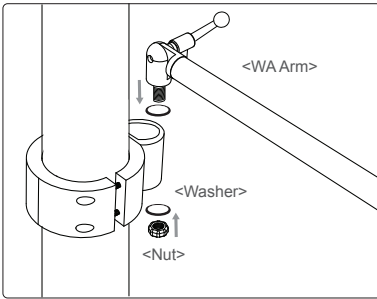

- Mount the WA Arm to WA Grip hole, and tighten up the nut tightly. (Put the washer between WA Arm and WA Grip, WA Grip and nut)

<How to use>

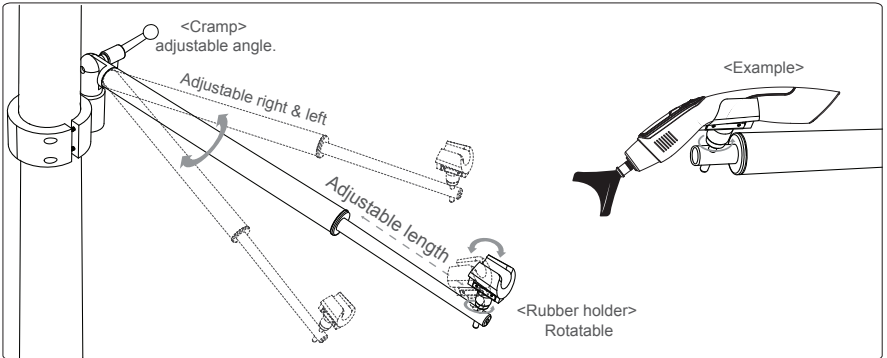
- Mount the device on the rubber holder.
- WA Arm is adjustable & left, angle, and length.
- Fit the angle, and tighten the WA Arm using by cramp.

Caution

- Read and check contents and manual before installation.
- Read and reference this manual. Careless on installation or of injury.
- Dropping this unit would lead to injury or device malfunction.
- Tighten kinds of bolt or nut surely to avoid accident.
- Check the bolt or nut whenever using to avoid accident.
- Manufacturer doesn't have any liability cause from misuse or user-careless.

6. How to use

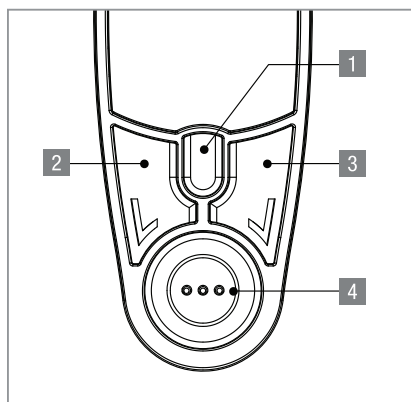
- 1) T/W mode button
 - Short click: Turbo mode
 - Long click: WA mode
- 2) Time button
 - Time select
- 3) Mode button
 - Mode select
 - * Normal-> High-> SoftStart-> Pulse-> PulseSoft
- 4) Operation button
 - Light irradiation
 - Power on/off

1. Activating handpiece

- Press the operation button gently. The LCD screen will be on standby mode and display the mode used last with a short audible tone.
- When battery is inserted, the device is automatically switched on.

Intro display

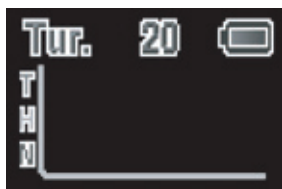Last used mode


2. Selection of the display mode

- Select the display mode (Graphic or Text) by pressing mode button for 1.5 sec. Beeps will indicate that the selection is complete.

<Graph Mode>

Graph Mode Display

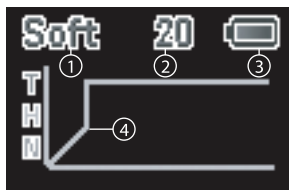
- ① Program mode
- ② Time
- ③ Battery
- ④ Output graph

6. How to use


<Text Mode>

Text Mode Display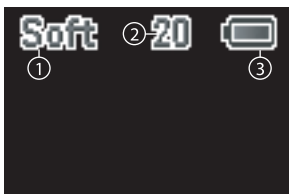

- ① Program mode
- ② Time
- ③ Battery

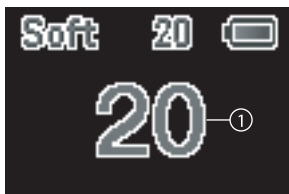- ① Remained time

Note

Initially, the selected curing time is displayed and subsequently the remaining time is displayed during the curing process.

3. Start of exposure

- Press operation button to start exposure with the selected mode.

Reference

Press any button if you wish to end the curing process.

4. Battery icon

- This icon indicates the power remaining in the battery.

- Recharge or replace the battery when "LOW BATTERY" sign appears on the display screen.

5. Sleep mode

- If unit is left unused for more than 5 minutes, all functions are automatically turned off with a short beep.

Reference

Press any button if you wish to end the curing process.

6. How to use

6. Set-up memory


- This handpiece will record the setting used last.

7. Selection of curing mode

- Six curing programs are available in this device.


1) Turbo mode

- This mode provides strong light intensity in a short time.
- The exposure time can be 5, 10, and 20 seconds.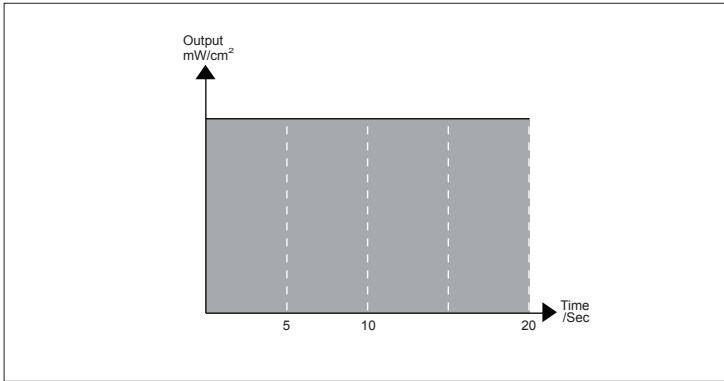

2) Normal mode


- This mode provides low light intensity for safety treatment.
- The exposure time can be 10, 20, and 30 seconds.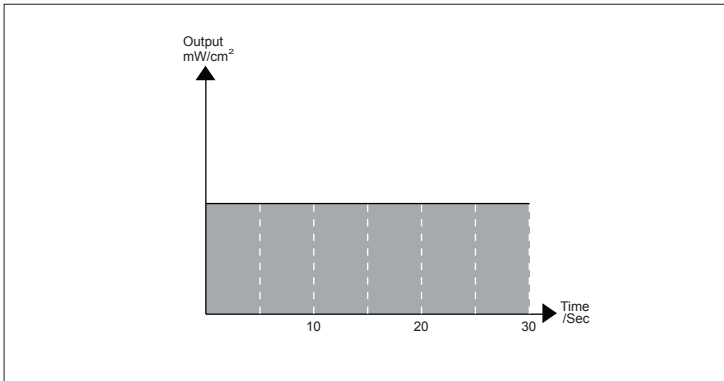

6. How to use


3) High mode

- The exposure time can be 10, 20, and 30 seconds.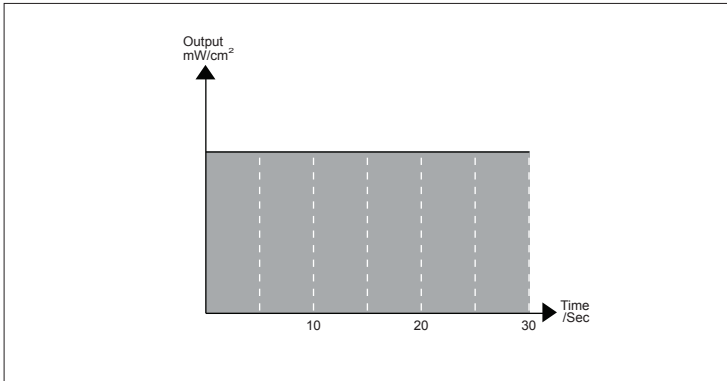

4) Soft Start mode


- The output power is raised from 0 for 5 seconds and then increased turbo mode power during the next 15 seconds.
- The exposure time can be 20 seconds. (fixed)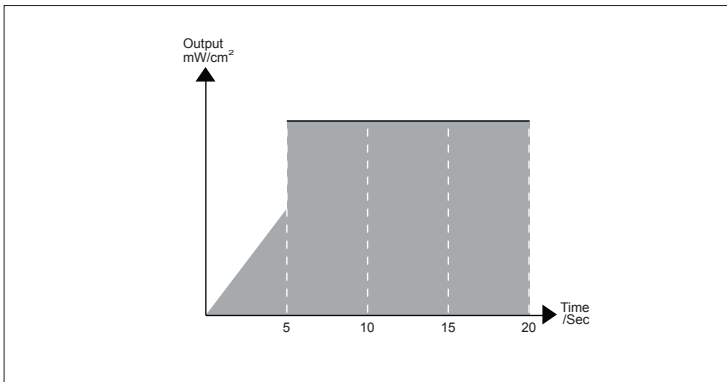

6. How to use


5) Pulse mode

- The output power start at turbo mode power and hold for 0.1 seconds followed by a rest period of 0.05 seconds in repetition.
- The exposure time can be 10, and 20 seconds.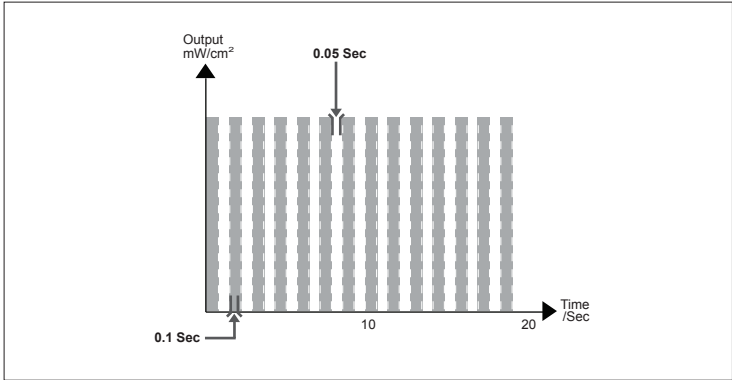

6) Pulse Soft Start mode

- Pulse and increased exposure from 0 to 5 seconds. And then increased turbo mode power during the next 15 seconds.
- The exposure time can be 20 seconds. (fixed)

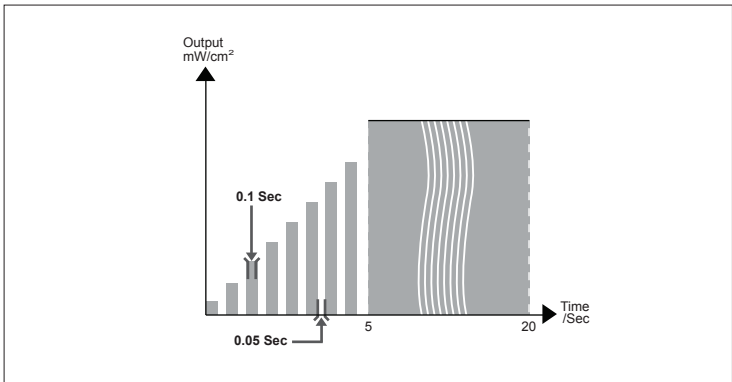6. How to use

7) WA (Whitening Accelerator) mode

- Change to WA tip
- Press T/W mode button for 2 seconds.
- Set the time with time button

* WA Mode intro display

* WA Mode processing display

Note

- To use WA mode, it is need WA Kit.
- Regarding whitening step, follow whitening gel instruction.

7. Maintenance

Cleaning and sterilization

- Avoid wetting the handpiece and charger to prevent electrical shock. Ensure that the charger and battery are always separated from the handpiece prior to cleaning.
- Clean the handpiece and charger gently with a clean, soft, lint-free cloth. Avoid using solvents containing oil or ethanol or acetone, sharp or pointed tools to clean as they may damage or scratch the plastic.
- Clean the guide tip with a soft cloth and disinfect. The guide tip is autoclaveable.
- Remove restorative materials stuck to the guide tip gently with a plastic spatula. Avoid using sharp tools as they may scratch the surface of the guide tip and affect the intensity of the light.

Maintenance and replacement of the battery

- Exposure to high room temperature may shorten the battery life. The recommended room temperature ranges between 15 to 25°C normal temperature. Extremes of room temperature will affect the operation of the LED light.
- It is recommended for the rechargeable battery to be charged immediately before use to maximize the efficiency.
- Remove the battery from the handpiece, if not being used for long time.
- If the running time declines with a full-charged battery, replace the battery before use.
- Exposure to high room temperature may shorten the battery life. The recommended room temperature ranges between 15 to 25°C normal temperature.

8. Trouble shooting

If the device is not operating normally, please refer to the following points.

Problem	Cause and solution
Buttons are not operating normally	<p>If the button is pressed with excess force, the power will automatically be switched off. In such a case press the button gently.</p> <p>In case it still does not operate normally, reconnect the battery to reset the device.</p>
Display screen does not light	<p>If the battery is completely discharged or connected incorrectly, display will not appear.</p> <p>Recharge the battery or reconnect the battery to reset the device.</p>
Power off while in operation	<p>If the operation button is pressed for more than 2 seconds, power will be switched off. Press the operation button again to restart the device.</p> <p>Check the battery connection.</p>
Not able to charge	<p>If the power cord is not correctly connected to the charger, or the handpiece is not properly placed on the charger.</p> <p>Clean the charger or the charging terminal.</p>
When intensity of light is poor	<p>If the guide tip has dirty or is damaged and the battery is not charged enough, intensity of light becomes poor. Replace with new guide tip or new battery.</p>

* If the trouble is not fixed after following solution, please contact your local dealer or manufacturer.

9. Specification

Equipment Class

Model:	Dr's Light
Safety:	IEC60601-1
EMC (Electro-Magnetic Compliance):	IEC60601-2
Protection from electric shock:	Type BF 
Protection from ingress of liquids:	IPX0 (ordinary equipment)
Operation:	Intended for continuous operation

Operating environment

Ambient temperature:	16°C to 40°C
Relative humidity:	10% to 80%
Atmospheric pressure:	500 hPa to 1060 hPa

Transport and Storage Environment

Ambient temperature:	-20°C to 60°C
Relative humidity:	10% R.H. to 85% R.H.
Atmospheric pressure:	500 hPa to 1060 hPa

Additional Information

Handpiece	Dimensions	Approx. 215 x 96 x 37 mm
	Weight	Approx. 175 g (including the battery)
Charger	Dimensions	Approx. 175 x 178 x 137 mm
	Weight	Approx. 225 g
Battery	Type	Lithium-ion battery
	Nominal voltage	DC 3.7V
	Dimensions	Approx. 107 x Max. H33mm
	Weight	Approx. 65 g

* this specification is subject to change without prior notice.

10. Accessories

- Guide tip
- Battery
- WA KIT

11. Warranty

This device is only shipped after strict inspection. Should the product incur any problems under normal use conditions within one year from the date of purchase, the repair will be free of charge. However, repair of consumables (Guide tip, Battery, and etc.) are not covered by the warranty. For more information, refer to the 12. Quality Assurance.

 Gruneworld GmbH

Georg-Weiss-Weg 9, 53125 Bonn, Germany
TEL: +49(0)288/7077695 FAX: +49(0)288/7077696
Website: www.gooddrs.de

Good Doctors Co., Ltd.

283, Bupyeong-daero, Bupyeong-gu, Incheon, Korea
(#208, B-dong, Woolim Lions Valley) (Zip code : 403-911)
Tel : + 82 32 424 6325 Fax : + 82 32 424 6326
E-mail : info@gooddrs.com Website : www.gooddrs.com

Good Doctors Good Doctors Good Doctors Good Doctors Good Doctors

We guarantee the quality of the product as follows

Name of product :

Name of model :

Customer information

- Name of dental clinic :

- Name :

Distributor

- Seller :

- Address :

- Telephone :

Assurance period

- Date of purchase : from _____ , 20____

Product is for 1 year except accessories.

< Assurance details >

1. If there is any defect of manufacturer or product has any fault itself with in the assurance period, it will be replaced with the same product.
2. Below cases are not effective for the free remuneration.
 - Malfunction and damage according to user's carelessness.
 - Malfunction and damage according to natural disaster such as fire, earthquake and flood etc.
 - Malfunction and damage according to user's intention or accident.
 - Malfunction and damage according to using incompatible product.
3. The manufacturer does not accept any liability resulting misuse.
4. The compensation method which compensate it based on consumer law.

Good Doctors Co.,Ltd.

Address : #208, B-Dong, Bupyeong Woolim Lions Valley, Cheongcheon 2-dong, Bupyeong-gu, Incheon, Korea (Zip code : 409-911)
Tel : +82 32 424 6325 Fax : +82 32 424 6326 E-mail : info@gooddrs.com Website : www.gooddrs.com

Good Doctors Good Doctors Good Doctors Good Doctors Good Doctors

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