**For dental use only**

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Symbol

MANUFACTURER		Certification mark	
Batch code		Date of manufacture	
Temperature limitation		Use by	
Consult instructions for use		Authorized representative in the european community	

han FIL

Light Curing Composite Resin

HanFil
 Light Curing Composite resin
 HanFil is a light curing composite resin.

Because it has a nano-filler, has good depth of cure, low shrinkage, low absorption, good strength, excellent mechanical properties and good bio-compatibility. It is suitable for esthetic tooth orthodontic and filling in all cavities case(Class I ~ V) because its hardness and abrasion are very similar to natural teeth.

■Cautions

1. HanFil must be handled by dentists and dental specialists according to the instructions before using it.
2. Recommend the Rubber dam for reduction of reaction for use with patients who have a history of severe allergic reaction to this material.
3. Do not use to patients who have rash on their skin, dermatitis or histories of hypersensitivity.
4. Stop applying this material to the patients who have symptoms of rash of hypersensitivity during using.
5. This material is used only to the oral tissue, Avoid contact with eyes.
6. If accidental contact occurs with eyes or skin, wash immediately with much water and seek medical advice.
7. Non-specialists must not use this material.
Do not use this material for other purposes.
8. Protective device for the respiratory tract is highly recommended before a surgical operation.
9. Do not use after expiration date(within 2years).

■Application Range

- Filling in vast range of anterior and posterior teeth (Class I to V)
- Restoration of an anterior tooth fracture
- Tooth orthodontic for esthetic purpose

■Application Method

1. Before the Application
 - 1) Check the expiration date and avoid package contamination.
 - 2) Follow the instruction for use before using it.
2. Application
 - 1) Select an appropriate shade with the patient's tooth.
 - 2) Prepare a cavity out of any contamination. Check the cavity whether remove of the amalgam and restorative resin or not.
 - 3) Etch all around cavities of enamel area with Etchant(HanEtch).
 - 4) Apply the adhesive agent(HanBond) on enamel and dentin area.
 - 5) Apply the composite resin(HanFil) on the cavities.
 - 6) Light cure for 20 seconds. Repeat the process 5) until tooth cavity is filled perfectly.
 - 7) Polish the surface of the filling material just filled.
3. Storage & Shelf-Life
 - 1) Close the safety cap immediately after use.
 - 2) Store this material in a shady, cool and dry place(2~27℃ / 35~80°F).

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