**⚠ For dental use only**

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## Symbol

MANUFACTURER		Certification mark	
Batch code		Date of manufacture	
Temperature limitation		Use by	
Consult instructions for use		Authorized representative in the european community	

**han FIL flow**

NANO based light curing flowable  
 composite resin 20 sec. curing time

## HanFil Flow

NANO based Light Curing Flowable Composite Resin  
 HanFil Flow is light curing flowable composite resin.  
 It has high flow ability and elasticity which is ideal for small cavity filling and could be applied in Cervical or Proximal. It has high hardness and elasticity make it easy to be used in both anterior and posterior tooth area.

## ■Cautions

- HanFil Flow must be handled by dentists and dental specialists according to the instructions before using it.
- Recommend the Rubber dam for reduction of reaction for use with patients who have a history of severe allergic reaction to this material.
- Do not use to patients who have rash on their skin, dermatitis or histories of hypersensitivity.
- Stop applying this material to the patients who have symptoms of rash of hypersensitivity during using.
- This material is used only to the oral tissue, Avoid contact with eyes.
- If accidental contact occurs with eyes or skin, wash immediately with much water and seek medical advice.
- Non-specialists must not use this material.  
Do not use this material for other purposes.
- Protective device for the respiratory tract is highly recommended before a surgical operation.
- Do not use after expiration date(within 2years).

## ■Application Range

- Filling of the small cavity
- Filling in vast range caries (Class I to V cavities)
- Restoration of the filling materials
- Tooth orthodontic for esthetic purpose
- Undercut block-out
- Filling of the micro-leakage
- Sealing of small pit and fissure
- Adhesion of the bracket
- Base/Liner of filling materials

## ■Application Method

- Before the Application
  - Check the expiration date and avoid package contamination.
  - Follow the instruction for use before using it.
- Application
  - Select an appropriate shade with the patient's tooth.
  - Prepare a cavity out of any contamination.  
Check the cavity whether remove of the amalgam and restorative resin or not.
  - Etch all around cavities of enamel area with Etchant(HanEtch).
  - Apply the adhesive agent(HanBond) on enamel and dentin area.
  - Apply the composite resin(HanFil Flow) on the cavities.
  - Light cure for 20 seconds. Repeat the process 5) until tooth cavity is filled perfectly.
  - Polish the surface of the filling material just filled.
- Storage & Shelf-Life
  - Waste the disposable tips.
  - Close the safety cap immediately after use.
  - Store this material in a shady, cool and dry place(2~27℃ / 35~80°F).

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