MODE D'EMPLOI
OPERATION INSTRUCTIONS
INSTRUCCIONES DE EMPLEO
ISTRUZIONI PER L'UZO
INSTRUÇÕES DE UTILIZAÇÃO
INSTRUKCJA OBSŁUGI
LÂGE FÖR SYSSLSÄTTNING
ΟΔΗΓΙΕΣ ΧΡΗΣΕΩΣ
GEBRUIKSAANWIJZING
ИНСТРУКЦИЯ ПО ЭКСПЛУАТАЦИИ

Français
English
Español
Italiano
Português
Polski
Svenska
Ελληνικά
Nederland
Русский

ENDY 6200

Sommaire

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Contents

Contents of the valise	6
Location and function of the controls	8 - 10
Using the Endy 6200 in continuous rotation	12 - 14
Connecting the contra angle	16
Fitting and removing the file	18
Fitting and removing the contra angle from its support ..	20
Mode selection	22
"Auto" mode	24
"Run" mode	26
"Apex over" mode	28
Selection and use of the memories	30
Adjusting the motor torque	32
Display of the force of your file	34
Adjusting the motor speed	36
Adjusting the apical stop	38
Adjusting the audible signal	40

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Contents

Important information about measurement	42
Information on the max speed / torque ratio	44
Use as an apex localiser	46
Connecting the file grip	48
Cleaning, disinfecting and sterilisation	50
Cleaning and lubrication of the contra angle	52
Periodical maintenance of the batteries	54
Charging	56 - 58
Replacing the battery pack	60 - 62
Accident prevention	64 - 72
Advice, problems and solutions	74 - 76
Technical characteristics, After Sales Service	78 - 86
Symbols	88 - 91
Electromagnetic emissions	92 - 97
Warranty	98
Warranty certificate	101

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Location of the controls

- 1 Memories
- 2 Indication of the torque selected and the force on the file
- 3 Indication of the speed selected
- 4 Reverse return indicator, motor / apex
- 5 Battery charge indicator
- 6 Battery indicator
- 7 Movement selection key
- 8 Adjustment selection key
- 9 Mode selection key
- 10 Indication of the mode selected
- 11 Indicator beep
- 12 Measurement error indicator
- 13 Display of the arrival at the apex
- 14 Reverse return indicator, motor / torque
- 15 Hand part holder

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Enter the pulp cavity, the micro-motor starts automatically. Progress slowly in the canal without forcing on the instrument; if the instrument stops in a canal and turns round (**Torque control**), exit from the canal. Start the sequence again or go to the next sequence (file change). The progression of your file towards the apex is shown by the luminous bar representing a canal. **Arrival at the buffer, the file stops automatically**; if the end of the file is at the buffer or beyond for one and a half seconds, **the rotation switches to slow reverse**.

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"Run" mode : Forced rotation, Apex auto-reverse (buffer), Torque auto-reverse.

The file rotates outside the pulp cavity. Penetrate slowly in the canal without forcing on the instrument; if the instrument stops in the canal and turns round (Torque control), exit from the canal. To change your file and go to the next sequence, **press the « $\vee$ » key to stop the motor** (the speed light will flash). To start your motor again, press the « $\wedge$ » **key for 1 second** (the speed light will become fixed again). The progression of your file towards the apex is shown by the luminous bar representing a canal. **At the buffer, the file stops automatically**; if the end of the file is at the buffer or beyond for one and a half seconds, **the rotation switches to slow reverse**.

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Enter the pulp cavity, the micro-motor starts automatically. Progress slowly in the canal without forcing on the instrument; if the instrument stops in a canal and turns round (Torque control), exit from the canal. Start the sequence again or go to the next sequence (file change). The progression of your file towards the apex is shown by the luminous bar representing a canal.

This mode does not stop the rotation at the apex (the buffer light is off). This mode should be used with caution and in cases where the measurement is upset by a damaged tooth, amalgam, etc.

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Selection and use of the memories

The ENDY 6200 has 12 memories. Each memory corresponds to a predefined torque and speed (Cf. T. 1). The apical stop buffer default setting is 0.5. The « ∇ » or « $\wedge$ » keys are used to change the memory. You can change the torque, the speed and the apical stop buffer of each memory (see the chapter entitled: torque, speed and apical stop buffer adjustment). The speed and torque values in the table are given for information only and may be changed.

- Read the file manufacturer's instructions.

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Adjusting the micro motor torque

To change the torque, press the « **Select** » **key once** (the Torque light will flash Fig. 1), the other parameters such as the speed, the buffer, etc. go out, then press the « **∨ or ∧** » key to increase or reduce the torque. Programming stops automatically after three seconds (the Torque light becomes fixed again and the other parameters such as the speed, the buffer, etc. light up again). Warning: this operation changes the torque value of the memory selected.

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Display of the force of your file

1 - Example: Your selected torque is « 2 ».

2 - As soon as your file starts turning the indicator light « 2 » for the torque selected goes out.

3 - The « Torque » light bar shows the force of your file in real time. Example 50% of the torque selected.

4 - The « Torque » light bar shows the force of your file in real time. Example 100% of the torque selected.

5 - If the maximum torque is exceeded, the indicator light of the torque selected « 2 » lights, and the blue light flashes indicating a torque auto-reverse.

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Adjusting the micro motor speed

Press the « **Select** » **key twice** in succession to select speed adjustment (Fig. 1 flashing of the speed light), the other parameters such as the torque, the buffer, etc. go off, then press the « **∨ or ∧** » key to increase or reduce the speed. Programming stops automatically after three seconds (the Speed light becomes fixed again and the other parameters such as the torque, the buffer, etc, light up again). Warning: this operation changes the speed value of the memory selected.

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Adjusting the apical buffer

Press the « **Select** » key **three times** in succession to select buffer adjustment (flashing of a light in the canal of your appliance), the other parameters such as the torque, the speed, etc. go off, then press the « **▼** or **▲** » keys to reduce or increase the buffer. Programming stops automatically after three seconds (the buffer light becomes fixed again and the other parameters such as the torque, the speed, etc. light up again). *Recommended buffer setting 0.5*

The buffer serves to stop the rotation of the instrument at a programmed distance before the apex. The distance can be programmed at 2.0 before the « Apex ». The default value on delivery of your appliance is 0.5 before the apex. Warning: this operation changes the buffer value of the memory selected.

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Adjusting the audible signal

To change the adjustment of the audible signal, press the « **Select** » **key four times** in succession to select the beep (flashing of a beep indicator light), the other parameters such as torque, speed, etc. go out, then press the « **∨ or ∧** » keys to reduce or increase the beep (a sound is emitted during the adjustment). Programming stops automatically after three seconds (the buffer light becomes fixed again and the other parameters such as the torque, the speed, etc. light up again). If the Beep is switched off, no light is alight (Fig1). Weak Beep (fig.2). Loud Beep (fig. 3).

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Connections and use of the Endy 6200 as a simple apex locator.

- 1 - Plug a white lead into the white socket on the left of your appliance.
- 2 - Plug the 2nd white lead into the blue socket on the left of your appliance.
- 3 - Connect a labial electrode to the end of a lead. Place the labial electrode on the patient's lip.
- 4 - Connect the file to the end of the 2nd lead.
- 5 - Press a key to start your appliance.

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- 6 - Press the « ✓ » key for 1 second** to stop the motor rotating. The speed indicator light flashes (fig. 1).
- 7 - Connect the file grip** to your manual file while pressing the back of the file grip.
- 8 - Start your** progression in the canal until you obtain a value close the apex (e.g.: 0.5). Set your stop, and measure the working length with a rule (not supplied).
- Note:** For use with the micro motor, don't connect the file grip and its lead to the blue sleeve.

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Cleaning, disinfection and sterilisation.

- Only use commercially available alcohol based colouring-free disinfectant (ethanol, isopropanol). The use of scouring powder or an abrasive sponge would damage the surface.
- The appliance is not protected against water splashes. It cannot be sterilised. Don't spray cleaner onto the control box or the motor.
- Sterilise the labial electrode, the file grip, the white leads and the contra angle before every use, only in an autoclave at 134°C (135°C maxi) for less than 20 minutes.
- Refer to the standards in force in your country for sterilisation methods.

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Cleaning and lubricating the contra angle before every use.

For lubrication with an aerosol *, insert the lubrication adapter into the rear of your contra angle (Fig. 1). Place the aerosol as shown in the sketch (Fig. 2). Spray for 2 to 3 seconds maximum. The oil flows through, cleans and lubricates the whole of the contra angle.

- The lubrication adapter cannot be put in the autoclave at 135° C.
 - Grease the contra angle before every sterilisation.
- * pharmaceutical hygienic sterilisation lubricant for dental hand tools.

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Periodical maintenance of the batteries

Autonomy: Approx. 2 hours with the micro motor. Automatic shut down of the appliance after 3 minutes of inactivity. Three lights at the bottom of your appliance indicate the state of charge of the batteries.

- 1 - Batteries charged > green light. .
- 2 - Batteries 50% charged > yellow light.
- 3 - Batteries discharged > red light.

- During vacation periods we advise charging the appliance once a fortnight using a programmer (not supplied).

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Charging your appliance:

- 1 - During charging two lights light up (one green and one orange).
- 2 - At the end of the charge the orange light goes off and the green light shows that charging is finished.
- 3 - If the battery is faulty a red light lights up instead of the two green and orange lights. Disconnect your charger, and then reconnect it. A new charging cycle starts. If the red light lights again, replace the battery pack.

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Accident prevention

Many maintenance and operating problems with the Endy 6200 are caused by a lack of attention to elementary safety precautions. Problems and accidents can be avoided by following the manufacturer's recommendations. Start by reading all cautions and instructions concerning safety and accident prevention thoroughly. Then run the equipment taking great care to prevent other damage to the equipment itself and to prevent injury.

IONYX is not responsible for accidents, material damage or injury resulting from:

- a repair by a person not authorised by IONYX.
- a change, modification or deterioration of the product.

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- the use of products or equipment produced by other manufacturers, except for those supplied by IONYX.
- maintenance or repair using spare parts or components other than those recommended by IONYX.
- operations other than those defined in the descriptive procedure of this manual or resulting from the non-observation of the precautions and accident prevention described in this manual.
- working, storage or installation conditions such as, for instance, a supply current not in conformity with the specifications.

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Users, that is dentists, are responsible for the maintenance and correct operation of this medical appliance. These medical appliances should only be used by members of the dental profession. IONYX does not approve any repairs carried out by a person not certified by IONYX.

Do not use this appliance with an electric scalpel or on a **patient wearing a cardiac stimulator (pacemaker)**. This appliance must not be connected to, or used in combination with, another appliance or system. IONYX is not responsible for accidents, material damage, injury or other problems resulting from ignorance of these clauses.

Never allow the appliance to fall or expose it to any type of impact or shock.

Precision in the measurement of the widening of the canal is not always possible depending on the shape and the condition of the tooth, as described in the user guide. Stop using the appliance immediately if you feel a tactile reaction suggesting abnormal conditions.

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Make sure that the file is fully home in its housing. Pull on the file slightly to make sure that it is properly fitted.

Never press the contra angle push button while the motor is running.

Take care not to injure your finger when inserting or removing the file from its housing.

Stainless steel files cannot be used with this appliance. Titanium nickel files require care to be taken about the points listed below to avoid breakages.

Before using a titanium nickel file, first enter the canal manually as far as the apical constriction with an ordinary file.

Never use force to insert the file into the contra angle.

Never use force to make the file enter the canal.

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Titanium nickel files can sometimes break as a result of wear. Don't use the same file for treating more than 10 canals. Maximum of 3 canals if there is significant curvature.
Follow the file manufacturer's directions: clinical protocol, duration of use, etc.
Never "jump" intermediate files. Files that are too big for the canals cause stoppage and reverse rotation of the motor.
Never use the same file to enlarge the same canal for more than 10 seconds.
Never use damaged files.
To work efficiently, you need to first acquire practice and experience. We recommend that the dentist practices on extracted teeth or root models.

DO NOT USE THE ENDY 6200 FOR OTHER SITUATIONS THAT ARE NOT SPECIFIED.

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Recommendations

Never use the apex location function with a patient wearing a cardiac stimulator (**pacemaker**). Only use the micro motor function. For this situation, remove the white lead and the labial electrode.

The use of hypochlorite higher than 2.5% can cause measurement errors at the canal entrance and during progression towards the apex.

Problems → Solutions

- **Measurement starts very close to the apical zone:**
 - Irrigate the canal with a conducting irrigation fluid (Dakin or Hypochlorite $\leq 2.5\%$ maximum)
 - Take a file that suits the size of the canal.

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Technical characteristics	
Manufacturer	Ionyx, 33290 Blanquefort - France
Model	ENDY 6200 - On the market in 2010
EN 60601-1 classification	Class B, Applied part of the BF, IP 30 type
93/42 CEE classification	Class IIa
Power source	3.6 Volt 900 mAh rechargeable battery pack 300 to 500 charge and discharge cycles Battery charger: 9 Volts 0.66 A
Contra angle (25g)	Maximum rotation speed: 600 rpm Diameter of tools 2.35 - type 1 Special coupling for ENDY 6200
Electrodes and contra angle	Autoclavable at 135° C
Length of leads:	Micro motor: 1.5 metres Measurement: 1.5 metres (Autoclavable at 135° C)
Control box	Made from white ABS with integral hand part rest
Weight / Dimensions in mm (HxLxD)	425gm / 97 x 186 x 80
Operating conditions	10°C to + 40°C 30% to 75% relative humidity
Storage and transport conditions	10°C to + 40°C 30% to 95% relative humidity

After Sales Service

For all maintenance or repairs, send your appliance to your distributor or directly to the Ionyx Service department (our address is on www.ionyx.eu). For a return under warranty please attach a copy of your invoice; this will help your request to be handled more quickly.

Before dispatching your equipment, clean, disinfect and sterilise it as explained in the user instructions.

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- For your appliance to receive a complete overhaul, please send the contra angle, the measurement leads, the file grip and the charger as well.

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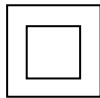
Electromagnetic emissions

1.6 Test Conditions and Results - Conducted Emissions

1.6 Test Conditions and Results - Conducted Emissions				
CISPR 11	TEST: Limits of mains terminal disturbance voltage			Verdict
Method: The AMN placed 0,8 m from the boundary of the unit under test and bonded to a ground reference plane. This distance was between the closest points of the AMN and the EUT. All other units of the EUT and associated equipment were at least 0,8 m from the AMN. All power was connected to the system through Artificial Mains Network (AMN). Conducted voltage measurements on mains lines were made at the output of the AMN.				P
Laboratory Parameters		Required prior to the test		During the test
Ambient Temperature		10 to 40 °C		22 °C
Relative Humidity		10 to 90 %		57%
Fully configured sample scanned over the following frequency range		Frequency range on each side of line		Measurement Point
		150kHz to 30MHz		Mains
Limits - Group 1 and 2 - Class B - EUT Configuration mode 2				
Frequency (MHZ)	Limit dB (uV)			
	Quasi Peak	Result *	Average	Result *
0,15 to 0,50	66 to 56	12	56 to 46	2
0,50 to 5	56	7,1	46	0,2
5 to 30	60	16,8	50	6,8
Supplementary information: The result in this table is the minimum margin to the limit.				

Electromagnetic emissions

1.7 Test Conditions and Results - Radiated Emissions		
CISPR 11	TEST: Limits for radiated disturbance 30 MHz - 1 GHz	Verdict
Method: Measurements were made in a 10-meter Open Area Test Site that complies to CISPR 16. Preliminary (peak) measurements were performed at an antenna to EUT separation distance of 10 meter. The EUT was rotated 360° about its azimuth with the receive antenna located at various heights in horizontal and vertical polarities. Final measurements (quasi-peak) were then performed by rotating the EUT 360 and adjusting the receive antenna height from 1 to 4 m All frequencies were investigated in both horizontal and vertical antenna polarity where applicable.		P
Laboratory Parameters	Required prior to the test	During the test
Ambient Temperature	10 to 40 °C	20 °C
Relative Humidity	10 to 90 %	57%
Fully configured sample scanned over the following frequency range	Frequency range on each side of line	Measurement Point
	30MHz to 1GHz	10 m measurement distance
Limits - Group 1 - Class B - EUT Configuration mode 1		
Frequency (MHZ)	Limit dB (uV/m)	
	Quasi Peak	Results *
30 to 230	30	4dB
230 to 1000	37	No frequency observed
Limits - Group 1 - Class B - EUT Configuration mode 2		
Frequency (MHZ)	Limit dB (uV/m)	
	Quasi Peak	Results *
30 to 230	30	3dB
230 to 1000	37	No frequency observed
Supplementary information: The result in this table is the minimum margin to the limit		

Electromagnetic emissions

1.8 Test Conditions And Results - Limits for Harmonic Current Emissions		
61000-3-2	TEST: Limits for Harmonic current emissions (IEC 61000-3-2: - use latest edition)	Verdict
Method: This test consists on the measurement of harmonics components of the input current which may be produced by equipment having an input current up to and including 16 A per phase, and intended to be connected to public low-voltage distribution systems. The equipment is tested under specified conditions of operation.		P
Laboratory Parameters	Required prior to the test	During the test
Ambient Temperature	15 to 35 °C	-
Relative Humidity	30 to 60 %	-
Equipment mode	Power interface mode	230V 50Hz
	EUT configurations mode	2
	Operation mode	2
Classification of Equipment		Class A
Supplementary information: This 10W equipment is compliant for harmonic current emissions without measurement. (p<75W)		

Electromagnetic emissions

1.9 Test Conditions and Results - Limitation of Voltage Fluctuations and Flicker		
61000-3-3	TEST: Limitation of Voltage Fluctuations And Flicker (IEC 61000-3-3: - use latest edition)	Verdict
Method: The test circuit consists of a test supply voltage, reference impedance, the equipment under test and a flicker meter compliant with IEC 60868 The equipment shall be tested in the condition in which the manufacturer supplies it.		P
Laboratory Parameters	Required prior to the test	During the test
Ambient Temperature	15 to 35 °C	-
Relative Humidity	30 to 60 %	-
Equipment mode	Power interface mode	230V 50Hz
	EUT configurations mode	2
	Operation mode	2
Supplementary information: This equipment is unlikely to produce voltage fluctuation and flicker. This equipment is compliant without		

Electromagnetic emissions

1.10 Test Conditions and Results - Immunity to Electrostatic Discharges			
61 000-4-2	TEST: Electrostatic discharges (IEC 61000-4-2:- use latest edition)		Verdict
Method: The test is intended to demonstrate the immunity of equipment subjected to static electricity discharges from operators directly and to adjacent objects. The tabletop equipment under test is placed on a 0.8 m on the table. The EUT and the cables are isolated from the coupling plane by an insulating support 0.5 mm thick. The floor standing equipment is isolated from the ground reference plane by an insulating support about 0.1 m thick. The vertical coupling plane (VCP) of dimensions 0.5 m x 0.5 m is placed parallel to, and positioned at a distance of 0.1 m from, the EUT.			P
Laboratory Parameters	Required prior to the test	During the test	
Ambient Temperature	15 to 35 °C	21 °C	
Relative Humidity	30 to 60 %	50%	
Test Levels			
Discharge type	Discharge Level (kV)		Number of discharges per location (each polarity)
	Positive	Negative	
Air-Direct	2,4,8	2,4,8	10
Contact-Direct	2,4,6	2,4,6	10
Contact-Indirect	2,4,6	2,4,6	10
Discharge type	See photo documentation of the test set-up All external locations accessible by hand Horizontal plate (HCP) Vertical coupling plate (VCP)		
Supplementary information: EUT powered at one of the Nominal input voltages and frequencies: 3.6V DC in configuration mode 1			

Electromagnetic emissions

1.11 Test Conditions and Results - Immunity to Radio Frequency Electromagnetic Fields				
61 000-4-3		TEST: RF electromagnetic fields (IEC 61000-4-3: - use latest edition)		Verdict
Method: The test allows estimating of the radiated immunity of electrical and electronic equipment to electromagnetic disturbances coming from intended radio-frequency (HF) transmitters in the frequency range 30 MHz to 1000 MHz. The interference is applied on the enclosure of the equipment by using transmitting antennas.				P
Laboratory Parameters		Required prior to the test		During the test
Ambient Temperature		15 to 35 °C		21 °C
Relative Humidity		30 to 60 %		49%
Test specifications				
Calibration Requirements		Uniform field area (UFA)	1.5 m x 1.5 m, 16 points with a minimum UFA size 0.5 m x 0.5 m	
			75 % of calibration points within specifications if UFA is larger than 0.5 m x 0.5 m. 100 % (all 4 points) in the specifications for 0.5 x 0.5 m UFA	
Frequency bandwidth		80 MHz to 2500 MHz		
Level	Non-Life Supporting Equipment	3V/m		
		Amplitude modulation	80 %/ 1 KHz sine wave	
			900 MHz and 1800 MHz Pulse at 80% AM - 200Hz	
			Controls, monitors or measures a physiological parameter, (80 % / 2 Hz)	
Level	Life Supporting Equipment	10V/m		
		Amplitude modulation	80 %/ 1 KHz sine wave	
			900 MHz and 1800 MHz Pulse at 80% AM - 200HZ	
			Controls, monitors or measures a physiological parameter, (80 % / 2 Hz)	
Frequency step		1% with 3s dwell time		
Supplementary information: EUT powered at one of the Nominal input voltages and frequencies: 3.6V DC in configuration mode 1.				

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Warranty

The warranty is valid for a period of one year starting from the purchase date of your product. All cases of failure to respect the recommendations in the operating instructions invalidate the warranty; in addition it does not cover wearing parts. Any work done on your appliance by a person not approved by ionyx invalidates the warranty. Damage linked to the non respect of the operating instructions or normal wear of parts are excluded from the warranty. For a return under warranty please attach a copy of your invoice; this will help your request to be handled more quickly. Before dispatching your equipment, clean, disinfect and sterilise it as explained in the user instructions. For quicker processing of your claim, please return the warranty certificate that was attached to this manual when you received your machine.

Wrap your appliance carefully before dispatching it; we cannot accept responsibility in the event of damage during transport.

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