HyFlex™CM

NiTi Root Canal Instruments

Instructions for use	EN
Indications for use: For dentin removal and root canal shaping.	

Instructions:

The spirals of the HyFlex CM files may lengthen in response to force. This prevents blockages in the canal and considerably reduces the risk of breakage. Unlike commercially-available NiTi instruments, the shape of the files can be restored. The files regain their original shape during autoclaving (or when a glass bead steriliser is used for 10 seconds).

It must be ensured that the spirals of the files do not wind in the opposite direction during use, as otherwise they will not regain their original shape. If several spirals of a file appear to have lengthened after autoclaving or if they appear to be faulty in any other way, the file should no longer be used (see Step by step card). The number of times a file can be re-used depends on processing and treatment. The condition of the files should always be verified before and after use.

- Process files prior to re-use (see instructions for processing resterilisable medical devices).
- A low-speed contra angle is required in order to use the files (e.g. Coltène Endo CanalPro CL motor handpiece). Operate the handpiece at 500 rpm (revolutions per minute). Recommended torque up to 2,5 N-cm (25 mN-m, 250 gcm).
- When using the files, irrigate the root canal frequently and ensure lubrication.
- Clean the spirals of the file every time after insertion into the root canal.
- Repeat after each step.

The files may be used with the crown-down and step-back techniques, or using the recommended hybrid and single length techniques described below.

Sterility:

HyFlex CM Root Canal Instruments are supplied presterilised using gamma irradiation. Sterility is guaranteed until use as long as the packaging has not been damaged or opened. Make sure to dispose of packages damaged prior to use.

Storage instructions:

HyFlex CM Root Canal Instruments should no longer be used after the expiry date.

Step by step instructions (see also the illustrated Step by step card):

Coronal Access

After straight-line coronal access has been established, it is useful to use a hand file (maximum size .02/20) to establish an apical glide path.

STEP 1:

Begin with the .08/25 file (orifice opener). Proceed forward slowly using tapping movements and without applying pressure. Proceed to Step 2 as soon as resistance is felt. Check patency using a .02/20 hand file. While doing so, ensure that the root canal always remains irrigated and lubricated.

Optional step: If the working length is reached in large canals it is possible to proceed directly to Step 4.

STEP 2:

Use the .04/20 file for apical enlargement up to the working length. Proceed forward slowly using tapping movements and without applying pressure. As soon as resistance is felt, check patency using a .02/20 hand

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COLTENE

file. While doing so, ensure that the root canal always remains irrigated and lubricated.

STEP 3:

Use the .04/25 file for apical enlargement up to the working length. Proceed forward slowly using tapping movements and without applying pressure. Return to the previous step as soon as resistance is felt. Check patency using a .02/20 hand file. While doing so, ensure that the root canal always remains irrigated and lubricated.

STEP 4:

Use the .06/20 file to shape the middle section of the canal until the working length is reached. Proceed forward slowly using tapping movements and without applying pressure. Return to the previous step as soon as resistance is felt. Check patency using a .02/20 hand file. While doing so, ensure that the root canal always remains irrigated and lubricated.

STEP 5:

Use the .04/30 file to enlarge the apical section until the working length is reached. Proceed forward slowly using tapping movements and without applying pressure. Return to the previous step as soon as resistance is felt. Check patency using a .02/20 hand file. While doing so, ensure that the root canal always remains irrigated and lubricated.

STEP 6:

Perform final preparation up to the working length using the .04/40 file. Proceed forward slowly using tapping movements and without applying pressure. Return to the previous step as soon as resistance is felt. Check patency using a .02/20 hand file. While doing so, ensure that the root canal always remains irrigated and lubricated.

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Coltène/Whaledent GmbH + Co. KG
Coltène/Whaledent Inc.