HyFlex™CM Crown-down NiTi Root Canal Instruments

Instructions for use	EN
----------------------	----

Indications for use:
For dentin removal and root canal shaping.

Instruccions:
The file can be used multiple times. The number of times a file can be re-used depends on processing and treatment. The condition of the files should always be verified before and after use. To reduce the risk of file separation a new file should be used for each procedure.

HyFlex CM Files may respond to pressure, torque and resistance with a lengthening of the spirals, which allows the file to avoid binding to the walls and therefore increases fracture resistance. Unlike normal NiTi instruments, this can be reversed. During autoclaving (or treatment in a glass bead heater for 10 seconds), the instruments can regain their shape. However, care has to be taken, if the instruments start to twist in the opposite direction during use, the files will not regain their shape and should be discarded. If after autoclaving, a file has multiple spirals which appear lengthened or otherwise seems not functional, the file should be discarded (see Step by step card).

- Use of a slow speed hand piece required. Operate the hand piece at 500 RPM (revolutions per minute). Recommended torque setting up to 2.5 N-cm / 25 mN-m / 250 gcm.
- Irrigate/lubricate the canal frequently when using the files.
- CanalPro NaOCl or CanalPro EDTA may be used as a lubricant/irrigant during rotary instrumentation.
- Clean file flutes after each insertion into the root canal.
- Recapitulate between each step.

Hyflex CM Files are available in various tip and taper sizes and can be used in Crown-down and Step-back techniques, or with the Hybrid technique. The following reflects a technique favoured by many clinicians:

Step by step instructions:

- Note:
- In most clinical situations three files will suffice for root canal shaping. Selection of the files to use should be determined by the clinician based on the individual case.
 - CanalPro NaOCl may be used as a lubricant/irrigant during rotary instrumentation.

Coronal Access

After gaining a Straight-Line coronal access, create an apical glide path to a 02/20 size with a hand file or with the HyFlex GPF.

STEP 1:
Start with the 08/25 file. Use as an orifice opener. Move slowly forward without pressure in a pecking motion. Once resistance occurs go to step 2. Check patency with 02/20 hand file and go back to the previous step, irrigate and lubricate.

STEP 2:
Depending on the size of the root canal use a 04/30 or 06/30 file to about 3-5mm short of working length. Move slowly forward without pressure in a pecking motion. In case resistance occurs use a 02/20 hand file to check patency. Irrigate and lubricate.

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Coltène/Whaledent GmbH + Co. KG
Coltène/Whaledent Inc.