



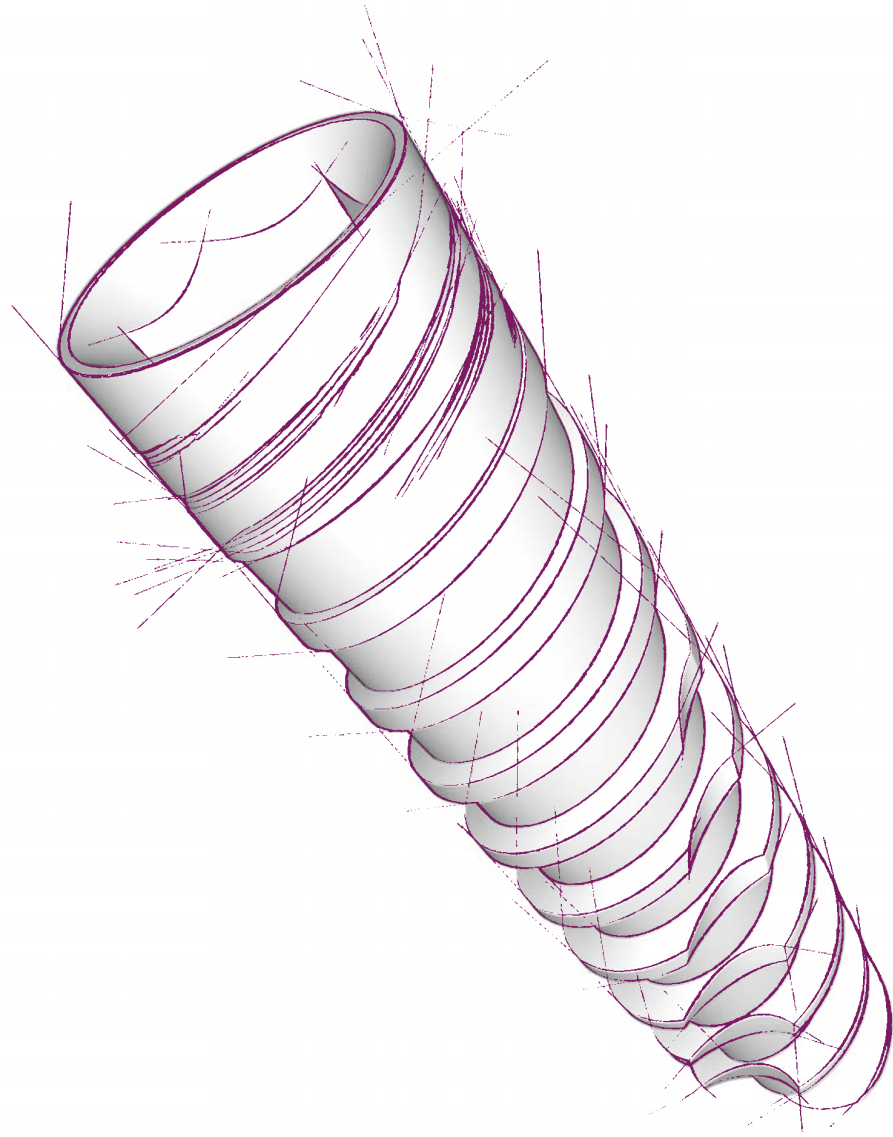

Dental Implant

JD EVOLUTION[®]S

SAFE AS ALWAYS, THINNER THAN EVER

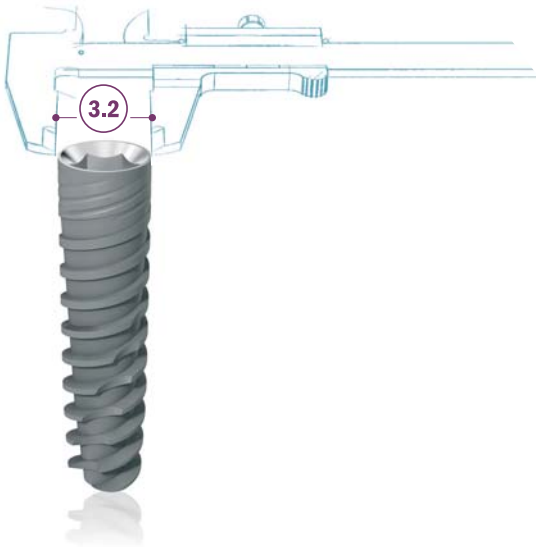


ADVANTAGES

- High primary stability, even in type 4 bone.
- Self drilling and self cutting.
- Redirecting capability for optimal placement.
- Immediate function capability.
- Built-in Platform switching.
- Solves tight spacing problems.
- Designed specifically for narrow spaces in the esthetic zone.

IMPLANT MEASURES AND LENGTHS

The JDEvolution®S dental implant is available in the lengths shown in the following chart:

Ø 3.2 mm					
8 mm	10 mm	11,5 mm	13 mm	15 mm	

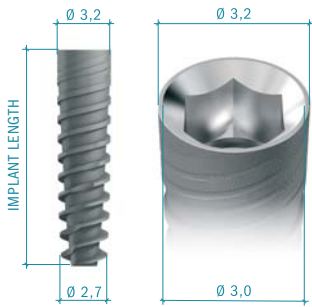

DRILLING SEQUENCE

It is recommended to adhere to the indications of the following drilling sequence to ensure optimal primary stability of the implant.

IMPLANT DIAMETER	SOFT BONE TYPE IV	MEDIUM BONE TYPE II-III	DENSE BONE TYPE I
Ø 3.2	1,5 (2)	2 2,4	2 2,4 (2,8)

Note: All dimensions are expressed in millimeters.

PRODUCT CATALOGUE

Implants:

Ø 3,2

EV32 080	JDEvolution® S Ø 3.2 L 8
EV32100	JDEvolution® S Ø 3.2 L 10
EV32115	JDEvolution® S Ø 3.2 L 11,5
EV32130	JDEvolution® S Ø 3.2 L 13
EV32150	JDEvolution® S Ø 3.2 L 15



Implant drivers:

ESID	Implant driver JDEvolution® S
ESIDL	Implant driver long JDEvolution® S



Cover screw:

ESCS	Cover screw JDEvolution® S
------	----------------------------



Healing Abutment:

ESHA323	Healing Abutment Ø 3.2 H 3.0 JDEvolution® S
ESHA325	Healing Abutment Ø 3.2 H 5.0 JDEvolution® S
ESHA327	Healing Abutment Ø 3.2 H 7.0 JDEvolution® S
ESHA403	Healing Abutment Ø 4.0 H 3.0 JDEvolution® S
ESHA405	Healing Abutment Ø 4.0 H 5.0 JDEvolution® S
ESHA407	Healing Abutment Ø 4.0 H 7.0 JDEvolution® S




Impression coping:

OPEN TRAY

ESICOT32C	Impression coping open tray Ø 3.2 JDEvolution® S
ESICOT40C	Impression coping open tray Ø 4.0 JDEvolution® S

CLOSED TRAY

ESICCT32C	Impression coping closed tray Ø 3.2 JDEvolution® S
ESICCT40C	Impression coping closed tray Ø 4.0 JDEvolution® S

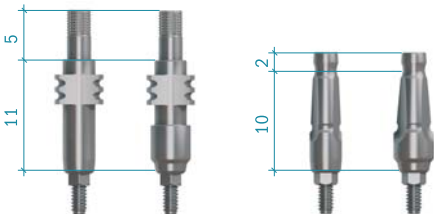

Temporary Abutment:

- ESTAEC

ESTANEC
- Temporary Abutment engaging JDEvolution® S

Temporary Abutment non engaging JDEvolution® S



Immediate Temporary Abutment:

- ESIT15C

EVITC
- Immediate Temporary Abutment h1.5 JDEvolution® S

Healing cap



GP Abutment:

- ESGPAE32C

ESGPAE40C

ES15GPAE40C
- GP Abutment Ø 3.2 JDEvolution® S

GP Abutment Ø 4.0 JDEvolution® S

GP Abutment 15° Ø 4.0 JDEvolution® S



Anatomic Abutment:

- ESAA15C

ESAA30C
- Anatomic Abutment Ø 4.0 H 1.5 JDEvolution® S

Anatomic Abutment Ø 4.0 H 3.0 JDEvolution® S



TiBase for Cerec Sirona:

- ESTIBC
- TiBase engaging JDEvolution® S



Wax up Abutment:

- ESWAEC

ESWANEC
- Wax-up Abutment engaging JDEvolution® S

Wax-up Abutment non engaging JDEvolution® S



Gold Abutment:

ESGAEC Gold Abutment engaging JDEvolution® S



Conical Abutment:

ESCA15C Straight Conical Abutment H 1.5 JDEvolution® S
ESCA1725C 17° Angulated Conical Abutment H 2.5 JDEvolution® S
ESCA3025C 30° Angulated Conical Abutment H 2.5 JDEvolution® S



Ball Abutment:

ESBA15 Ball Abutment H 1.5 JDEvolution® S
ESBA30 Ball Abutment H 3.0 JDEvolution® S
ESBA50 Ball Abutment H 5.0 JDEvolution® S
EVBAA Ball Abutment replica JDEvolution® S
EVBAHC Cap attachment housing and elastic retentive cap Ball Abutment
EVBAN Elastic retentive cap Ball Abutment



Emi Abutment:

ESEMI15 Emi Abutment H 1.5 JDEvolution® S
ESEMI30 Emi Abutment H 3.0 JDEvolution® S
ESEMI50 Emi Abutment H 5.0 JDEvolution® S
EVEMIA Emi abutment replica JDEvolution® S
EVEMIH C Cap attachment housing and elastic retentive cap Emi Abutment
EVEMIN Elastic retentive cap Emi Abutment



Prosthetic screw:

ESS Abutment Screw JDEvolution® S



Implant replica:

ESAN Implant replica JDEvolution® S





JDENTALCARE

www.jdentalcare.com